



Operating Instructions for 6875 Chair Scale

- ① Scale operator presses **ON/OFF** key. Display shows zero weight.
- ② Place patient on scale. Display shows patient's weight. If desired, lock patient's weight by pressing the **LOCK/RELEASE** key. The Lock annunciator appears in the display to indicate the Locked condition.
- ③ Remove patient from scale. Scale returns to zero. If display has been locked in step two, patient's weight will remain in display. Press **LOCK/RELEASE** key to return display to zero.
- ④ If scale is not used for two minutes and display remains at zero, the scale enters the sleep mode. The sleep mode is indicated by the display showing *SLEEP*. The scale must stay at a zero weight to remain in the sleep mode. While in the sleep mode, weighing operations may resume simply by placing a patient on the scale. Proceed as in step two.
- ⑤ If the scale is not used for seven more minutes after entering the sleep mode (nine minutes from when the last patient was weighed), the scale will turn off automatically and the display will become blank. Return to step one.

INDICATOR SETTINGS

Auto Shutoff = 9 Minutes
Sleep Mode = 2 Minutes
Lock Feature = Disabled



0065-C238-1A Rev G
08/06

CARDINAL SCALE MFG. CO.
PO Box 151 • Webb City, MO 64870
Ph: 417-673-4631 • Fax: 417-673-5001
www.detectoscale.com

Printed in USA

Technical Support: Ph: 866-254-8261 • techsupport@cardet.com



Operating Instructions for 6875 Chair Scale

- ① Scale operator presses **ON/OFF** key. Display shows zero weight.
- ② Place patient on scale. Display shows patient's weight. If desired, lock patient's weight by pressing the **LOCK/RELEASE** key. The Lock annunciator appears in the display to indicate the Locked condition.
- ③ Remove patient from scale. Scale returns to zero. If display has been locked in step two, patient's weight will remain in display. Press **LOCK/RELEASE** key to return display to zero.
- ④ If scale is not used for two minutes and display remains at zero, the scale enters the sleep mode. The sleep mode is indicated by the display showing *SLEEP*. The scale must stay at a zero weight to remain in the sleep mode. While in the sleep mode, weighing operations may resume simply by placing a patient on the scale. Proceed as in step two.
- ⑤ If the scale is not used for seven more minutes after entering the sleep mode (nine minutes from when the last patient was weighed), the scale will turn off automatically and the display will become blank. Return to step one.

INDICATOR SETTINGS

Auto Shutoff = 9 Minutes
Sleep Mode = 2 Minutes
Lock Feature = Disabled



0065-C238-1A Rev G
08/06

CARDINAL SCALE MFG. CO.
PO Box 151 • Webb City, MO 64870
Ph: 417-673-4631 • Fax: 417-673-5001
www.detectoscale.com

Printed in USA

Technical Support: Ph: 866-254-8261 • techsupport@cardet.com