


I N S T R U C T I O N S



Feeding Bottle

Catalog Number 74578-0000

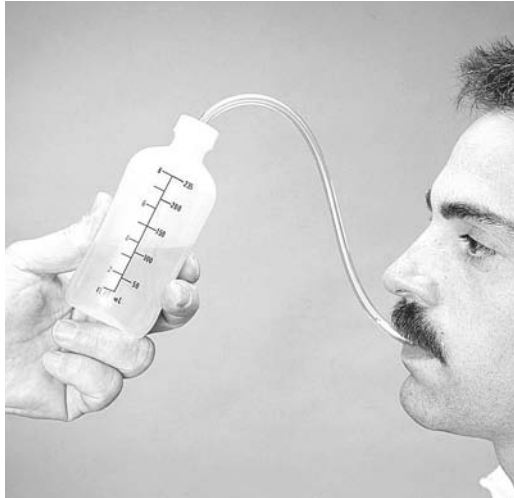

Useful for individuals who have difficulty chewing, sucking or swallowing. Can be used during oral motor treatment sessions to improve drinking skills. A tube extending from the bottom of the bottle out through the screw cap helps control placement and amount of liquid intake and will accommodate any lip closure. Bottle is graduated in ounces and milliliters to aid in dispensing measured amounts of liquid. Holds 8 oz. (237 ml).

To Use:

Fill the bottle with a thin liquid such as water, juice, milk, etc. Screw the cap on tightly. Place the end of the tube in the patient's mouth.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA
973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com