

75339-0000

# All-Out™ Universal Opener

Makes it easier and less painful for people with arthritis and other hand limitations to open packages, containers, bottles and cans

- Provides extra leverage to make opening packages and bottles simple and pain-free
- Comfortable and easy to hold
- Designed to accommodate several different grasping positions
- Soft, flexible T-shaped handle reduces the amount of grip needed



Box Tops



Pull Tabs



Twist Caps




Crimped Bottle Caps

## Instructions for Use:

### Opening box tops:

1. Slide pointed end of the opener (1) between box flaps
2. Slide opener across to break the seal

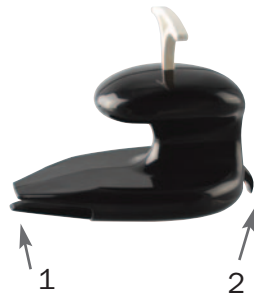

### Opening pull tabs:


1. Place hook (2) through the hole in the pull tab
2. Pull up until the tab is raised from the can
3. Reposition hook (2) to go through the front of the pull tab then pull back on the opener

### Opening twist caps:

1. Place bottle opener (3) over cap and twist

### Opening crimped bottle caps/non-twist caps:

1. Slide bottle cap into slot (4)
2. Pull opener up and away from the bottle

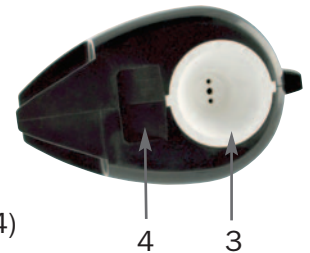

*"Special Products for People with Special Needs"*