

I N S T R U C T I O N S



Pediatric Seat

Catalog Number 70443-0000

This lightweight and portable seat has two different heights to accommodate the growth of your child. By adding a strap (not provided) the seat can be transformed into a booster seat – a highchair alternative. Non-skid feet prevent the unit from slipping.

To use: Choose the appropriate height for your child. This can be determined by the individual comfort of the child.

NOTE: The child's feet should be able to touch the ground.

Straps: Each side of the seat has slots to accommodate straps with a width up to 1" (2.54 cm). Simply slip the strap or straps through the bottom of the seat and attach securely around the child. If two straps are used it is important that each strap is secured to the bottom of the seat and will not slip completely through the slots.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com