

I N S T R U C T I O N S



Multi-Level Hand Grip

The Multi-Level Hand Grip is available in three sizes:

Single Level	Cat. No. 72481-0001
Double Level	Cat. No. 72481-0002
Triple Level	Cat. No. 72481-0000

Thank you for purchasing the Ableware® Multi-Level Hand Grip. When properly installed, it will provide an extra measure of security and safety to shower, bathtub and toilet walls. Vertical installation on a shower or tub wall provides a safe support for adults and children. Installation near toilets aids the infirm or elderly in sitting down and rising from the toilet seat.



Please Note: The loops of the Hand Grip have grooved inner surfaces for a non-slip grasp; nonetheless, it is strongly recommended that you do not grasp it with soapy hands.

Caution: Proper installation is important! This Hand Grip is designed for vertical installation only. The Hand Grip should be mounted to a wall stud. Installation requires a drill, drill bits, silicone caulking, and a screwdriver. DO NOT use an adhesive to mount the Hand Grip to the wall. For any other type of installation, consult a professional.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com

972481002

0602

I N S T R U C T I O N S



Mounting Instructions:

1. Locate a wooden wall stud, hold the Hand Grip in the desired location, and mark the holes with a sharpened pencil.
2. Use a 1/8" drill bit to drill holes. NOTE: For tile-surfaced walls, use 3/16" masonry drill to pass through the tile only; then use a 1/8" drill.
3. Apply silicone caulking on the pads of the Hand Grip to prevent water seepage into screw holes.
4. Hold Hand Grip in place and match up holes.
5. Starting at the top end of the Hand Grip, insert screws into holes and tighten them firmly into the stud with screwdriver.
6. Press in screw covers.

Caution: DO NOT put entire body weight on the Hand Grip. It is intended to be a support aid only.

Cleaning:

A mild detergent on a sponge is recommended. Do not use any abrasive substance.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com

972481002

0602