

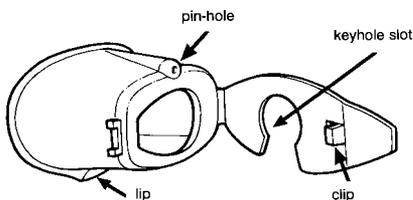
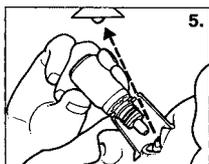
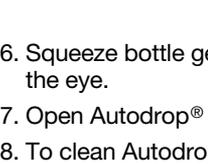
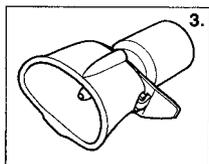
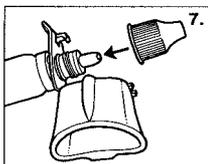


# I N S T R U C T I O N S

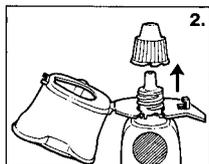

## Autodrop® Eye Drop Guide

Catalog Number 78677-0000

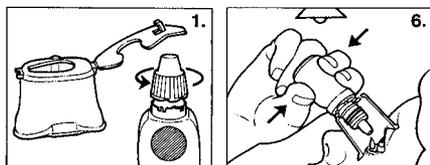

1. Open Autodrop®. Loosen cap from bottle, but do not remove completely.
2. Place the keyhole slot around the base of the bottle neck.
3. Close the Autodrop®. Do not force the bottle nose into Autodrop®, it should close easily.
4. Hold down lower eyelid. Place Autodrop® over the eye with the lip of the Autodrop against the cheek.
5. Tilt head backwards, look through pin-hole towards the light.
6. Squeeze bottle gently to allow prescribed drops to fall into the eye.
7. Open Autodrop® and replace top on eye drop bottle.
8. To clean Autodrop® remove eye drop bottle and wash Autodrop® in warm soapy water. Rinse and dry thoroughly.

We recommend that Autodrop® is used for the whole course of treatment for an eye disorder. When an eye drop bottle is discarded remove Autodrop® and clean thoroughly before fitting to the new bottle. Autodrop® may be used on more than one type of eye drop at the same time but if you suffer from more than one eye complaint we recommend that you use a separate Autodrop® for each type of eye drop.

For clinics and hospitals: Autodrop® may be autoclaved on a low pressure cycle for 30 minutes. Release clip to allow moisture to evaporate. Autodrop® can also be cleaned in sterilizing fluid. Rinse and dry thoroughly.

## Maddak

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email [custservice@maddak.com](mailto:custservice@maddak.com) • [www.maddak.com](http://www.maddak.com)