



DETECTO IS EMR-READY!

With increased interest in connecting Detecto's scales to Electronic Medical Records software, it is important to know the true advantages of Detecto's power in the market place.

Send weight, time, date, BMI (Body Mass Index), and height directly to a printer, PC, or network.




758C Indicator Front

THE POWER OF DETECTO'S 758C!

All Detecto scales featuring the 758C indicator are EMR ready. The new 6437DHR (shown to the left) is a great example. The 6437DHR is designed for optimum performance and displays height, weight, and BMI simultaneously by simply raising the digital height rod while weighing a patient.


No Keys To Push! No Operational Sequences To Memorize! Fully Automatic!

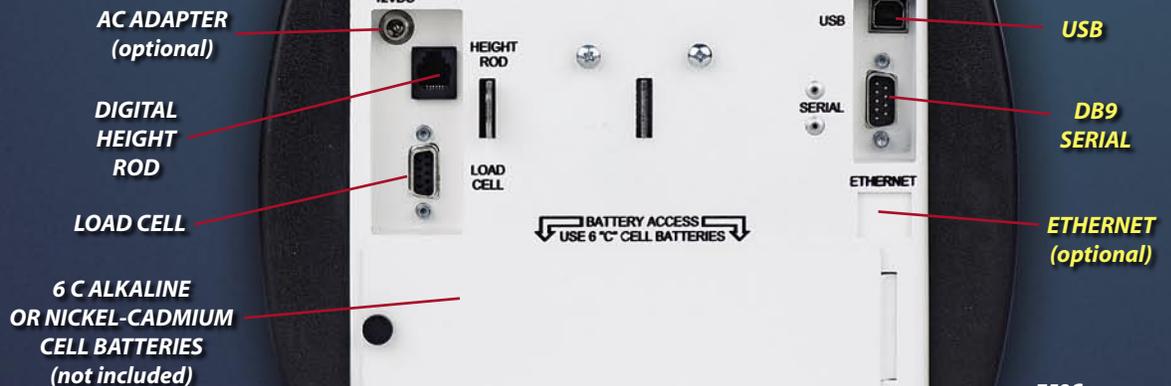

758C Indicator Back

758C CONNECTIVITY

All Detecto scales featuring the 758C indicator are EMR ready. The 758C includes a DB9 serial connector, USB port, and optional Ethernet card for those applications that require the scale to be on a computer network.

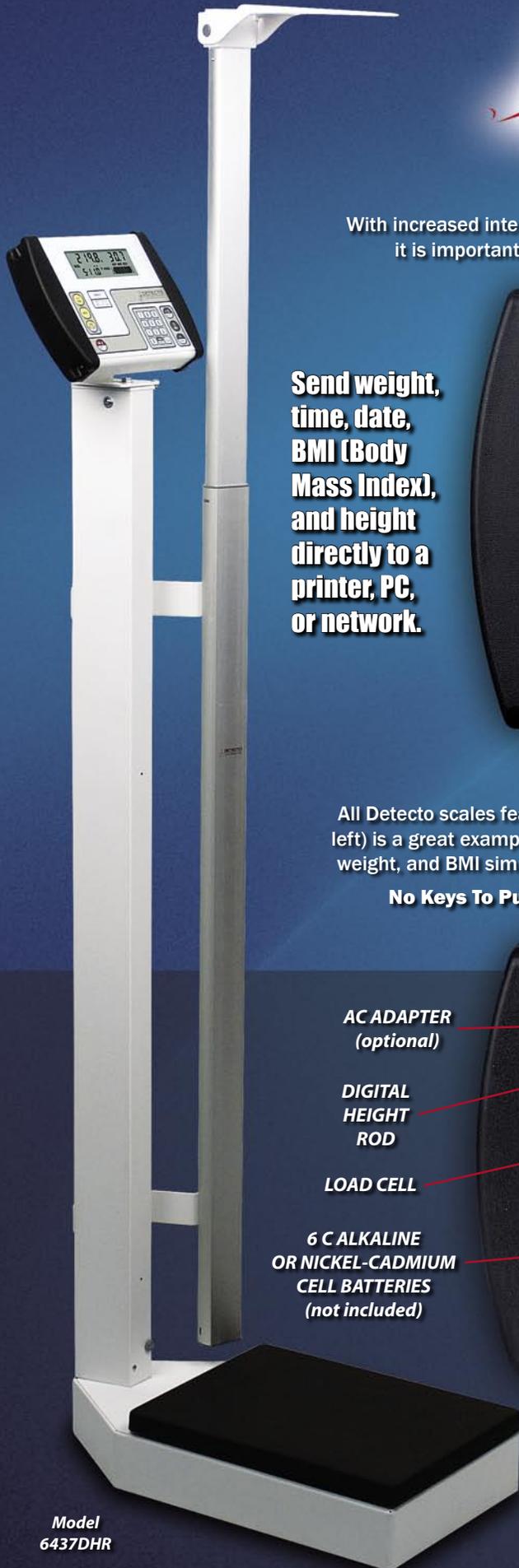

Model 6437DHR



DETECTO®

A Division of Cardinal Scale Manufacturing Co.



TRANSMITTING DATA WITH THE 758C

Detecto's 758C supports several formats and techniques for transmitting data. Data transmission may be initiated via computer command or operator input and transmitted via the RS-232, USB or Ethernet ports.

The simplest protocol is the P185 Journal format. See pages 26 and 28 of the 758C manual. When the patient is standing on the scale, adjust the digital height rod, and either press the PRINT key or invoke the Journal Print command from the connected computer. This will transmit data only when a the patient's weight/height needs to be recorded and requires very little processor time for the EMR, ensuring that the patient is on the scale with the height rod properly placed.

The following data/information will be sent via the selected port:

Time, Date, ID, BMI, Height, Weight <CR><LF>

- Time= hh:mm
- Date= mm/dd/yy
- ID= xxxxxxxxxxx (patient ID)
- BMI= xx.x (percentage)
- Height= x' xx.x" or xx.x cm (inches or centimeters)
- Weight= xxx.xuu G (gross weight)

ASCII Output String:

hh:mm,mm/dd/yy,xxxxxxxxxx,xx.x,x' xx.x",xxx.xuu G<CR><LF>



Model
6857DHR

The new 6857DHR is designed for bariatrics and nursing homes. It includes a low-profile, high-capacity scale base and handrails for patients who require support while weighing.


USA-MANUFACTURED DESIGN QUALITY

If a particular EMR software package requires a custom data format, Detecto Scale can provide a special quotation to output the weight record in a special format. This is all made possible since Detecto is a complete 100% software/hardware manufacturer located in Webb City, Missouri. No other medical scale company can do it like we do it... because WE MAKE IT!



Detecto Scale reserves the right to improve, enhance, or modify features and specifications without prior notice.

SOLD BY:

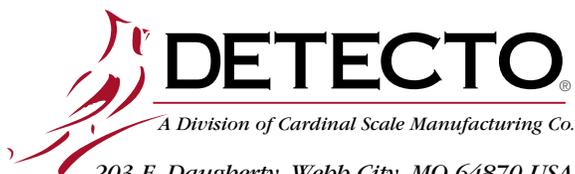

DETECTO®

A Division of Cardinal Scale Manufacturing Co.

203 E. Daugherty, Webb City, MO 64870 USA

Pb: 417-673-4631 or 1-800-641-2008 Fax: 417-673-5001

www.Detecto.com